

SMRT[®] Link software installation guide (v13.0)



Research use only. Not for use in diagnostic procedures.

P/N 103-009-000 Version 02 (January 2024)

© 2024 Pacific Biosciences of California, Inc. ("PacBio")

Information in this document is subject to change without notice. PacBio assumes no responsibility for any errors or omissions in this document.

Certain notices, terms, conditions and/or use restrictions may pertain to your use of PacBio products and/or third party products. Refer to the applicable PacBio terms and conditions of sale and to the applicable license terms at pacb.com/license.

Trademarks:

Pacific Biosciences, the PacBio logo, PacBio, Circulomics, Omnione, SMRT, SMRTbell, Iso-Seq, Sequel, Nanobind, SBB, Revio, Onso, Apton and Kinnex are trademarks of Pacific Biosciences of California Inc. (PacBio).

See https://github.com/broadinstitute/cromwell/blob/develop/LICENSE.txt for Cromwell redistribution information.

PacBio 1305 O'Brien Drive Menlo Park, CA 94025 www.pacb.com



SMRT[®] Link software installation guide (v13.0)

Introduction

This document describes the procedure for installing **SMRT Link v13.0** or **SMRT Link Lite v13.0**. This document is for Customer IT or SMRT Link administrators.

- SMRT Link v13.0 supports Revio[™] systems, Sequel[®] II systems and Sequel IIe systems. Sequel systems are not supported.
- SMRT Link Lite v13.0 supports Revio systems and Sequel IIe systems only. Sequel and Sequel II systems are not supported.

SMRT Link is the web-based end-to-end workflow manager for PacBio long-read systems. It includes software applications for designing and monitoring sequencing runs, and analyzing and managing sequence data.

SMRT Link is the primary access point for applications used by researchers, laboratory technicians, instrument operators, and bioinformaticians. The applications include:

- Instruments: View information about systems connected to SMRT Link.
- **Sample Setup**: Calculate binding and annealing reactions for preparing DNA libraries for use on **all** supported systems.
- **Runs**: View information about sequencing runs, monitor run progress, status and quality metrics, design sequencing runs and create and/or import run designs.
- **Data Management**: Create Projects and Data Sets; manage access permissions for Projects and users; generate QC reports for Data Sets; view, import, export, or delete sequence, reference, barcode and target regions files.
- SMRT[®] Analysis: Perform secondary analysis on the basecalled data (such as sequence alignment, variant detection, structural variant calling, and RNA analysis) after a run has completed.
 Note: The SMRT Analysis module is not included when you install SMRT Link Lite.

Note: SMRT Link, SMRT Link Lite, the Revio system, Sequel II systems and Sequel IIe systems are for **research use only** (RUO).

Overview

- 1. Install or upgrade the SMRT Link software. (See "Installation summary" on page 7 and "Upgrading SMRT Link" on page 10 for details.)
- 2. (**Optional**) Configure SMRT Link or SMRT Link Lite to use an SSL certificate. (See "Installing an SSL certificate for NGINX" on page 21 for details.)
- 3. (**Optional**) Add SMRT Link Users and Assign User Roles. (See "Adding SMRT Link users via LDAP integration and assigning user roles" on page 18 for details.)
- 4. (**Optional**) Change the admin and pbicsuser passwords. (See "Changing admin and pbicsuser passwords" on page 13 for details.)
- 5. (**Optional**) Configure LDAP. (See "LDAP integration" on page 14 for details.)

Compute requirements for Revio systems and Sequel IIe systems

Network connection					
Connection 1 GbE or 10 GbE copper					
	Shared data storage				
	Per Revio system	Per Sequel Ile system			
Sequencing data ^a	72 TB/year	20 TB/year			
SMRT Link server					
	SMRT Link	SMRT Link Lite			
CPU cores	16	4			
RAM	64 GB	16 GB			
Local storage	1 TB SSD	500 GB SSD			
	HPC node (SMRT Link only)				
CPU cores	64				
RAM	4 GB per core, 256 GB total				
Local storage	Local storage 100 GB SSD or HDD (500 GB preferred as future workflows may us more local disk space.)				
Recommended	d HPC nodes by application ^b (SMRT	Link only)			
	Per Revio system	Per Sequel Ile system			
Human genome	2 nodes	1 node			
Single-cell transcriptome	2 nodes	1 node			
Large gene panels	1 node	1 node			

a. Per Revio system. Assumes standard hifi_reads.bam file without kinetics. Does not include data files produced in data analysis, which approximately doubles the storage requirement.

b. Per Revio system at utilization of 1,300 SMRT Cells per year. Compute is proportional to the number of SMRT Cells run. For example, half the number of nodes is needed at 650 SMRT Cells per year compared to 1,300 SMRT Cells per year. The number of nodes assumes that analysis is performed as data is generated throughout a year.

Note: Single-system compute configurations are available – contact your PacBio Bioinformatics Field Application Specialist (FAS) for details.

Data storage

• The SMRT Link software's installation **root** directory **must** be readable and writable by the SMRT Link install user (\$SMRT_USER) and **must** be addressable along the same installation path (\$SMRT_ROOT) on **all** relevant cluster nodes via NFS. PacBio recommends /opt/pacbio/smrtlink for the SMRT Link software's installation root directory (referred to as \$SMRT_ROOT), and smrtanalysis for the SMRT Link install user (referred to as \$SMRT_USER).

- The SMRT Analysis job output directory is used to store output from SMRT Analysis jobs. The software accesses this directory through a symbolic link (\$SMRT_ROOT/userdata/jobs_root) that points to the desired job output directory location. The link can be modified by using the installation script. The symbolic link destination should be on a shared file system (NFS); it must be writable by the \$SMRT_USER, and it must be addressable along the same path on all compute nodes. The default is to keep these output directories on the same NFS export as the SMRT Link installation, but optionally may be symbolically linked to a larger storage volume.
- The SMRT Analysis **database** directory is used to store database files and backups. The software accesses this directory through a symbolic link (<code>\$SMRT_ROOT/userdata/db_datadir</code>) that points to the desired database directory location. The link can be modified by using the installation script. This symbolic link destination should be a **local** directory (**not** NFS) and be writable by <code>\$SMRT_USER</code>. This directory should exist **only** on the SMRT Link install host.
- The SMRT Analysis **temporary** directory is used for fast I/O operations during run time. The software accesses this directory through a symbolic link (<code>\$SMRT_ROOT/userdata/tmp_dir</code>) that points to the desired temporary directory location. The link can be modified manually or using the installation script. This symbolic link destination should be a **local** directory (**not** NFS), it must be writable by <code>\$SMRT_USER</code>, and the link destination must exist (or be creatable) as an independent directory on **both** the head node and the compute nodes.

Software prerequisites: Server operating systems and pre-installed software

- SMRT Link server software is supported on:
 - English-language CentOS 7.x, supported until end-of-life 6/30/2024.
 - English-language Rocky Linux 8.x supported until end-of-life 5/31/2029.
 - Ubuntu 20.04 and 22.04 64-bit Linux® distributions.
 - These supported versions **also** apply to SMRT Link compute nodes.
- Note: PacBio advises against new installations of CentOS for use with SMRT Link.
- SMRT Link is **not** guaranteed to work on Linux versions that are no longer supported by the operating systems' vendors.
- SMRT Link server software cannot be installed on systems running other versions of UNIX, macOS[®] or Windows[®].
- Singularity v3.10.5 or later is **required** for the **Variant Calling** and **HiFi Target Enrichment** analysis workflows.
 - We recommend installing Singularity as root, with the setuid bit enabled (chmod u+s singularity) in the /bin or /usr/bin directory.
 - If you run jobs with a scheduler (such as SLURM), we recommend installing the singularity-ce package on the SMRT Link server, as well as each of the nodes within the relevant partition that you are submitting jobs to.
 - Singularity should **not** be installed in an NFS area of the file system.
 - The singularity binary cannot be installed to any file system area mounted with the nosuid and/or noexec mount options.
 - To run Variant Calling or HiFi Target Enrichment, Singularity will need to download several Docker images (docker://google/deepvariant:1.5.0; docker://google/deepvariant:1.5.0-gpu; docker:// quay.io/biocontainers/whatshap:1.4--py39hc16433a_1; docker://ghcr.io/dnanexus-rnd/ glnexus:v1.4.1 and docker://broadinstitute/picard:2.27.5). To save these images locally, we provide a script, \$SMRT_ROOT/admin/bin/fetch-singularity-cache, which uses the root account to download the images, using /tmp as a temporary file space, before depositing the .sif files into \$SMRT_ROOT/userdata/singularity and changing ownership to the requesting user. We recommend running this script after installing SMRT Link.
 - For further details on configuring Cromwell for Singularity, see here.

Software/hardware prerequisites: Client systems

To use SMRT Link on a client operating system:

- SMRT Link requires the Google® Chrome web browser, version 90 or later.
- SMRT Link **requires** a minimum screen resolution of 1600 by 900 pixels.

Network configuration

- Refer to the Site prep guide provided with your instrument purchase for more details.
- For network connectivity considerations, see the network diagram in the **Network requirements** section of the **Site prep guide**.

SMRT Link server environment assumptions

- The SMRT Link server should run on a dedicated 64-bit Linux host with libc 2.17 or greater.
- The installation is performed by the **same** non-root user (\$SMRT_USER) that will be used to run the SMRT Link web services.
- The <code>\$SMRT_USER</code> has full permissions recursively throughout the install directory, and in all linked directories for <code>jobs_root</code>, <code>db_datadir</code> and <code>tmp_dir</code>. (Common problems include NFS setup problems, ACLs, and so on.)
- When running in distributed mode, all other nodes have the **same path** for \$SMRT_ROOT and for all linked directories. (The NFS exports should have identical mount points on **all** cluster nodes.)
- No other daemons/services processes are bound to the same ports as the SMRT Link services.
- PacBio highly recommends that the system clock be synchronized to a domain or public NTP time server.
- The \$SMRT_USER service account must have both the nofile and nproc soft user limits set to a minimum of
 8192. (See the ulimit(1) and limits.conf(5) Linux man pages for more information.)
- The host operating system must provide the en_US.UTF-8 locale/character set.
- SMRT Link and SMRT Link Lite are not designed to handle changes in the hostname. If you are using them to connect to a Revio instrument you should ensure that the configured hostname is and will remain accessible across the network.

SMRT Link database note

- SMRT Link v13.0 no longer includes weekly automatic database backups. A database backup is still
 automatically performed once, during installation or upgrade. Failure to back up the SMRT Link database on
 a regular schedule risks losing all records in SMRT Link (including users, Data Sets, analyses, barcodes, and
 references) if a file system or reconfiguration error occurs. The underlying sequencing or analysis files, such
 as BAM files, are not affected.
- We **strongly** recommend asking your local Linux system administrator to schedule regular weekly backups of the SMRT Link database using standard Linux utilities. A utility script to generate an appropriate cron server command was added at \$SMRT_ROOT/admin/bin/generate-cron-backup. For additional details, please contact PacBio Technical Support.

General security notes

- PacBio **recommends** that you install the SMRT Link server on networks that are only accessible to **trusted** users, and discourages installing SMRT Link on public networks.
- Do not install SMRT Link or run SMRT Link services as the root user.

SMRT Link v13.0 security notes

SMRT Link v13.0 restricts access to the web services API to clients running on localhost (such as the API gateway that handles authentication and permissions) or remotely using SSL encryption and password-based authentication.

Support for the WSO2 API Manager was **deprecated** in v13.0, and this API will **no longer** be supported for new features. New installations will **automatically** start the replacement API gateway. **Customers with Revio systems who are upgrading existing SMRT Link installations will need to migrate to the new API gateway**

before Revio instrument connections can be added. No changes to connected Revio/Sequel II systems/Sequel IIe systems are necessary.

Ports and firewalls: SMRT Link end users **must** be able to access the SMRT Link server on port 8243. This port is also used by the Instrument Control Software (ICS), so it must be accessible to **any** Revio systems, Sequel II systems or Sequel IIe systems as well.

- If your network configuration already allows access to port 8243, **no additional changes** are required to use SMRT Link v13.0 or SMRT Link Lite v13.0.
- The instrument must have access to TCP port 8243 of the SMRT Link server.
- End users must also have access to TCP port 8243 of the SMRT Link server for access to the browser UI.
- Communication between Revio instruments and SMRT Link is bidirectional, so SMRT Link **must** have access to TCP port 9243 on any associated Revio instruments.

SMRT Link Lite v13.0

SMRT Link Lite v13.0 is a modified configuration that uses the **same** installer and software as SMRT Link v13.0, but with the most compute-intensive components (SMRT Analysis) disabled to support running on non-server hardware. Without these components, SMRT Link Lite can comfortably run on laptops with at least 16 GB RAM available, and supports configuring and displaying run results for a **single** Revio instrument.

Installation/upgrade checklist

Following is a list of items you should have ready **before** starting a new installation or upgrading an existing installation. **Note**: Paths that include spaces are **not** supported.

- Full path to the \$SMRT_ROOT directory.
- A service account (called the *\$SMRT_USER* in this document) to install and run the web services.
- Full path to the installation root directory, used for the main installation root.
- · Job Management System settings.
- Full path to a directory on the shared file system the jobs_root directory.
- Full path to a directory on the local file system on each node the tmp_dir directory.
- Full path to a directory on the local file system on the install node the db_datadir directory.
- (Optional) LDAP settings. See "Configuring LDAP in Keycloak" on page 14 for details.
- (Optional) SSL certificate for NGINX. See "SMRT Link and SSL certificate procedures" on page 21 for details.

Installation summary

Following are the steps for installing SMRT Link v13.0 or SMRT Link Lite v13.0 on a **new** system. (See "Appendix A: SMRT Link workflow terminology" on page 24 for details.)

• To upgrade SMRT Link to v13.0 from a **previous version**, see "Upgrading SMRT Link" on page 10.

SMRT Link v13.0 and SMRT Link Lite v13.0 can be used with the following supported version of ICS:

- v13.0 for Revio systems. Note: SMRT Link Lite works only with ICS v13.0.
- v11.0+ for Sequel II systems and Sequel IIe systems.

SMRT Link installation options

The following table lists the types of SMRT Link installations and what they include:

Installation type	GUI	Command- line tools	JMS integration	Sample data	Barcode and reference files	SMRT Link services	Cromwell	Cromwell with call caching
Full SMRT Link	Y	Y	Y	Y	Y	Y	Y	Y
SMRT [®] Tools only	Ν	Y	Ya	Ν	Ν	Ν	Y	Ν
SMRT Link Lite	Y	Y	Ν	Y	Barcode only	Y	Ν	Ν

a. JMS integration on a SMRT Tools-only installation may be setup using pbcromwell. See SMRT Tools reference guide (v13.0) for more information.

Step	Installation summary - SMRT Link v13.0/SMRT Link Lite v13.0				
1	Download SMRT Link software: Download and extract the SMRT Link software installer from here. (The same installer can install both SMRT Link and SMRT Link Lite.)				
2	Definitions and variables: For clarity, this document uses these conventions to refer to site-specific information:				
	• \$SMRT_ROOT: The SMRT Link Install Root Directory, such as /opt/pacbio/smrtlink.				
	• \$SMRT_USER: The SMRT Link Install User, such as smrtanalysis.				
	• smrtlinkhost.mydomain.com: The fully-qualified domain name of the SMRT Link Install Host.				
	 smrtlinkhost: The short host name of the SMRT Link Install Host. 				
	For <code>\$SMRT_ROOT</code> , defining the variable in the shell allows the commands below to be run verbatim. To do so, use something like:				
	SMRT_ROOT=/opt/pacbio/smrtlink				
	The fully-qualified domain name of the SMRT Link Install Host may always be used in place of the short host name. But in some cases, particularly when working with WSO2, the fully-qualified domain name is required.				
3	Log onto the SMRT Link Install Host (such as the hostname or IP address) as the SMRT Link Install User (such as \$SMRT_USER.)				
4	Install SMRT Link by invoking the SMRT Link Installer:				
	./smrtlink_ <version number="">.runrootdir \$SMRT_ROOT</version>				
	Note : The <code>\$SMRT_ROOT</code> directory must not exist when the installer is invoked, as the installer will try to create it, and will abort the installation if an existing <code>\$SMRT_ROOT</code> location is found.				
	If a previous installation was canceled or otherwise failed, the installer can be invoked without extraction. Rerun using theno-extract option:				
	./smrtlink_ <version number="">.runrootdir \$SMRT_ROOTno-extract</version>				
	Alternatively, install SMRT Link Lite by using the following command instead:				
	./smrtlink_ <version number="">.runlite truejmstype NONErootdir \$SMRT_ROOT nworkers 4</version>				
	See "Appendix A: SMRT Link workflow terminology" on page 24 for additional information.				
5	On the instrument, click Install when prompted to install the Chemistry Update.				
6	Start SMRT Link services: \$SMRT_ROOT/admin/bin/services-start				
7	(SMRT Link only) Run the Site Acceptance Test from the command line:				
-	\$SMRT_ROOT/admin/bin/run-sat-services				
	Successful completion of run-sat-services indicates that the HPC configuration is functioning correctly. This creates a "PacBio Example SAT Job" analysis entry in the SMRT Analysis section of the SMRT Link GUI.				

Step	Installation summary - SMRT Link v13.0/SMRT Link Lite v13.0
8	(Optional) Clear the browser cache:
	This is a good troubleshooting step if needed.
	 Open the Chrome Browser and choose More Tools > Clear browsing data, choose All Time from the Time Range control, then check Cached images and files. Click Clear data.
	2. Restart the browser.
9	(Optional) Configure LDAP and/or add local users:
	See "Configuring LDAP in Keycloak" on page 14, "Adding local users to SMRT Link using Keycloak" on page 18 for details.
10	(Optional) Configure SMRT Link/SMRT Link Lite to use a signed SSL certificate:
	See "Installing an SSL certificate for NGINX" on page 21 for details.
11	(Optional) Change the admin and pbicsuser passwords:
	We recommend that you change the admin and pbicsuser account passwords from the default values. See "Changing admin and pbicsuser passwords" on page 13 for details.

Upgrading SMRT Link

Supported upgrade path

- SMRT Link upgrades to v13.0 are supported from any v8.x, v9.x, v10.x, v11.x or v12.x releases.
- SMRT Link v13.0 can be used with the following supported version of ICS:
 - v13.0 for Revio systems
 - v11.0+ for Sequel II systems and Sequel IIe systems

Step	Upgrading SMRT Link
1	Download SMRT Link software: Download and extract the SMRT Link software installer from here.
2	Definitions and variables: For clarity, this document uses these conventions to refer to site-specific information:
	 \$SMRT_ROOT: The SMRT Link Install Root Directory, such as /opt/pacbio/smrtlink. \$SMRT_USER: The SMRT Link Install User, such as smrtanalysis.
	• smrtlinkhost.mydomain.com: The fully-qualified domain name of the SMRT Link Install Host.
	• smrtlinkhost: The short host name of the SMRT Link Install Host.
	For \$SMRT_ROOT, defining the variable in the shell allows the commands below to be run verbatim. To do so, use something like:
	SMRT_ROOT=/opt/pacbio/smrtlink
	The fully-qualified domain name of the SMRT Link Install Host may always be used in place of the short host name. But in some cases, particularly when working with WSO2, the fully-qualified domain name is required.
3	Log onto the SMRT Link Install Host (such as the hostname or IP address) as the SMRT Link Install User (such as <code>\$SMRT_USER.</code>)
4	Stop the SMRT Link services:
	<pre>\$SMRT_ROOT/admin/bin/services-stop</pre>
	Note: Ensure that no active SMRT Link analysis jobs are running before stopping services.
5	Upgrade SMRT Link by invoking the SMRT Link installer:
	./smrtlink_ <version number="">.runrootdir \$SMRT_ROOTupgrade</version>
	Note: The \$SMRT_ROOT directory must be an existing SMRT Link installation. Several validation steps will occur to ensure that a valid \$SMRT_ROOT is being updated.
	If a previous upgrade was canceled or otherwise failed, the installer can be invoked without extraction. Rerun using theno-extract option:
	./smrtlink_ <version number="">.runrootdir \$SMRT_ROOTupgradeno-extract</version>
	See "Appendix A: SMRT Link workflow terminology" on page 24 for additional information.
6	On the instrument, click Install when prompted to install the Chemistry Update.
7	Start the SMRT Link services:
	<pre>\$SMRT_ROOT/admin/bin/services-start</pre>
	If you are upgrading from an installation that used WSO2 API Manager, you must migrate to the new API gateway in order to connect to Revio systems:
	<pre>\$SMRT_ROOT/admin/bin/services-startmigrate</pre>
	The migration launches an interactive CLI tool after the server starts; as this is a new installation very few steps are required. Once migration is finished, SMRT Link automatically starts with the new API gateway in the future.

Step	Upgrading SMRT Link			
8	Run the Site Acceptance Test from the command line:			
	<pre>\$SMRT_ROOT/admin/bin/run-sat-services</pre>			
	Successful completion of run-sat-services indicates that the HPC configuration is functioning correctly. This creates a "PacBio Example SAT Job" analysis entry in the SMRT Analysis section of the SMRT Link GUI.			
9	(Optional) Clear the browser cache:			
	This is a good troubleshooting step if needed.			
	 Open the Chrome Browser and choose More Tools > Clear browsing data, choose All Time from the Time Range control, then check Cached images and files. Click Clear data. 			
	2. Restart the browser.			
10	(Optional) Change the admin and instrument user passwords:			
	We recommend that you change the admin and pbicsuser account passwords from the default values. See "Changing admin and pbicsuser passwords" on page 13 for details.			

Updating the SMRT Link Chemistry and UI Bundles

SMRT Link Bundle updates allow updating of SMRT Link features **without** having to reinstall the SMRT Link software. As of SMRT Link v13.0, there are two Bundle types for which an update indicator may appear:

SMRT Link Chemistry Bundle

 This includes kit and DNA Control Complex names used in the Sample Setup and Runs modules. The update also updates Instrument Control Software (ICS).

SMRT Link UI Bundle

• This includes changes and fixes to the SMRT Link graphical user interface (GUI).

Updating the SMRT Link Bundles from SMRT Link

Note: Only SMRT Link users with the **Admin** role can perform these updates. In addition, SMRT Link **must** have a route to the internet and update services **must** be enabled.

- 1. In SMRT Link, choose **Settings > Updates**. (A red circle indicates that one or more Bundle Update(s) are available.)
- 2. Click the **Update Chemistry Bundle** button.
- 3. Click the Update UI Bundle and Restart UI Server button, if applicable.
- 4. If there are any problems, clear the browser cache: Choose More Tools > Clear browsing data, choose All Time from the Time Range control, then check Cached images and files. Click Clear data. Refreshing the browser tab or clearing the browser cache may be necessary for the Bundle update(s) to take effect.

Updating the SMRT Link Chemistry Bundle on the instrument

The SMRT Link Chemistry Bundle will also need to be upgraded with the same Chemistry Bundle files used by the SMRT Link web services. Once SMRT Link's Chemistry Bundle is updated, the updates are passed along to any PacBio instruments configured to use that same instance of SMRT Link. Once available, follow the instructions below to update the Chemistry Bundle on the instrument side.

- 1. On the instrument, choose **Admin** from the Main menu. (A red circle indicates that a Chemistry Bundle Update is available.)
- 2. Click the **Updates** tab, then click **Install**. The instrument software then restarts, which will take around 10 minutes.
- 3. Click the **question mark** to check the version number to validate the Chemistry Bundle update.

Rolling back a SMRT Link UI Bundle update

SMRT Link UI Bundle updates allow updating of SMRT Link UI features **without** having to reinstall the SMRT Link software. This includes changes and bug fixes to the SMRT Link graphical user interface. When you upgrade the SMRT Link UI by clicking **Setting > Updates > Update UI Bundle and Restart UI Server**, the previous version of the UI is saved to a time-stamped folder. For example:

\$ ls -1 \$SMRT_ROOT/current//bundles/smrtlink-analysisservices-gui/current/private/pacbio/smrtlinkanalysisservices-gui/tomcat_current/webapps/ROOT/ -rw------ 1 fas Domain Users 158 Jan 22 12:11 index.jsp -rw-r--r-- 1 fas Domain Users 36780 Oct 4 15:16 pacbio-manifest.json -rw-r--r-- 1 fas Domain Users 10357 Oct 4 15:16 pacbio-manifest.txt drwx------ 7 fas Domain Users 4096 Oct 10 08:30 sl drwx------ 7 fas Domain Users 4096 Oct 10 08:30 sl_20200103_135348 lrwxrwxrwx 1 fas Domain Users 20 Oct 4 15:16 version.json -> pacbio-manifest.json lrwxrwxrwx 1 fas Domain Users 19 Oct 4 15:16 version.txt -> pacbio-manifest.txt

In this example, s1 is the tomcat_current/webapps/ROOT root directory installed by the bundle update, and s1_20200103_135348 is the backup of the original UI code.

To **downgrade** to the previous UI version, follow these steps:

- 1. Stop the server: \$SMRT ROOT/admin/bin/services-stop.
- 2. Rename the sl directory to any unique, recognizable name.
- 3. Rename the backup directory to sl.
- 4. Start the server: \$SMRT_ROOT/admin/bin/services-start.

Skipping a SMRT Link UI Bundle update

To skip a specific SMRT Link UI Bundle update, use the pbservice skip-update command:

\$ pbservice skip-update smrtlink-ui --user admin --password admin --host smrtlink-uri -port 8243
Marked bundle update smrtlink-ui/9.0.0.99999 as ignored. Note that you will need to re-login to the
SMRT Link UI for this to take effect.

\$ pbservice skip-update smrtlink-ui --user admin --password admin --host smrtlink-uri -port 8243
No upgrades found for bundle smrtlink-ui

To **undo** skipping the bundle, run an identical command using unskip-update. Note that the UI will still show the pending update unless you log out and log in again (the bundle's update status is cached upon initial login.)

Installing only SMRT Tools

To install **only** command-line SMRT Tools, use the --smrttools-only switch when calling the installer, whether for a new installation or an upgrade. (This installs the **same** command-line tools as a full installation.) Examples:

```
./smrtlink-11.1.0.xxxxx.run --rootdir smrtlink --smrttools-only
./smrtlink-11.1.0.xxxxx.run --rootdir smrtlink --smrttools-only --upgrade
```

Note: Using --smrttools-only will **only** unpack the command-line applications, and will **not** run through the configuration prompts or provide the web services of a full SMRT Link installation. If command-line only use with JMS integration is desired, see the **SMRT Tools reference guide (v13.0)** on how to setup JMS integration using pbcromwell.

Warning: Revio systems, Sequel II systems and Sequel IIe systems **cannot** communicate with a --smrttools-only installation.

Updating the SMRT Link Chemistry Bundle on smrttools-only installations

Use this procedure only if you have installed the SMRT Link package using the --smrttools-only switch.

Download the Chemistry Bundle from the PacBio website, then unpack the files and place them in a userdefined directory. The value of the <code>\$SMRT_CHEMISTRY_BUNDLE_DIR</code> environment variable then defines where the software finds the updated files. Following are the suggested best practices for installing the Chemistry Bundle:

- 1. Download the Chemistry Bundle from the PacBio web site's **Software Downloads** page.
- 2. (Optional) Define \$SMRT_ROOT for convenience: SMRT ROOT=/opt/pacbio/smrtlink
- 3. Make directories, unpack, and link:

```
mkdir -p $SMRT_ROOT/userdata/chemistry/chemistry-pb-13.0.0.xxxxx
tar -C $ SMRT_ROOT/userdata/chemistry/chemistry-pb-13.0.0.xxxxx -xf /path/to/chemistry-pb-
13.0.0.xxxxx.tar.gz
ln -s ./chemistry-pb-13.0.0.xxxxx $SMRT ROOT/userdata/chemistry/chemistry-pb-active
```

4. Define and export the \$SMRT CHEMISTRY BUNDLE DIR environmental variable and validate:

export SMRT_CHEMISTRY_BUNDLE_DIR=\$SMRT_ROOT/userdata/chemistry/chemistry-pb-active

5. Define the <code>\$SMRT_CHEMISTRY_BUNDLE_DIR</code> environment variable in the appropriate startup script for your shell to make it permanent. Example: Use ~/.bashrc.

Changing admin and pbicsuser passwords

The SMRT Link admin account has full access to SMRT Link, and is used to create users and grant users access.

SMRT Link comes with a default Instrument Control Software (ICS) user account (pbicsuser) which is used by the Revio system, Sequel II system, and Sequel IIe system to communicate with SMRT Link web services over a secure, encrypted connection. The pbicsuser account is **required** for instruments to communicate with SMRT Link. (Note that the pbicsuser credentials can **only** be used to access SMRT Link resources – it is **not** an LDAP account or a local account on the Linux system.)

The passwords for the admin and pbicsuser accounts are set to default values that are the same for **all** SMRT Link installations. Because the passwords can be used to access SMRT Link accounts and information, the passwords should be changed and only given to **trusted** users who require access.

To change the built-in account passwords for the new API gateway, use the following procedure while the server is running:

```
$SMRT_ROOT/admin/bin/set-keycloak-creds --user admin --password `NEW-PASSWORD' --admin-password
`CURRENT-PASSWORD'
$SMRT_ROOT/admin/bin/set-keycloak-creds --user pbicsuser --password `NEW-PASSWORD'
```

To verify the admin and pbicsuser passwords, use the following procedure:

\$SMRT_ROOT/smrtcmds/bin/pbservice status --host localhost --user admin --ask-pass \$SMRT_ROOT/smrtcmds/bin/pbservice status --host localhost --user pbicsuser --ask-pass

The pbservice status information should display, before exiting with an exit status of 0 indicating success.

Sequel II/Sequel IIe users must **also** change the pbicsuser account password in the Instrument Control Software (ICS) to match the new password. To do so: Select **Menu > Admin > SMRT Link** on the instrument touch screen to change the password.

Note: You can also use the Keycloak Admin interface on HTTPS port 9443 to change these passwords. (Unlike in previous versions of SMRT Link, changing the admin account password or even the user name with the user management interface now works correctly.)

Changing usage tracking settings

When first logging in to the SMRT Link GUI after a successful installation or upgrade, users are prompted to notify PacBio of the upgrade/installation success and whether they wish to share SMRT Link analysis usage information with PacBio. Once set, these settings may **only** be viewed and modified from the command line using the accept-user-agreement tool.

WARNING: To use the accept-user-agreement tool, services must be running:

\$SMRT_ROOT/admin/bin/services-start

To set new settings, use the following command, specifying true or false for the options accordingly. For example:

\$SMRT_ROOT/admin/bin/accept-user-agreement --install-metrics true --job-metrics true

PacBio is notified of a successful installation or upgrade immediately if the install metrics setting is true.

To view the current settings, run the command without any arguments:

\$SMRT_ROOT/admin/bin/accept-user-agreement

Note: If accept-user-agreement is run **without** arguments and the settings have not been previously set (either in the GUI or on the command line), **both** the install and job metrics settings will automatically be set to true and PacBio will be immediately notified of the installation or upgrade.

Starting SMRT Link automatically on server boot

To start SMRT Link automatically when the server boots using systemd, refer to the template service file located here:

```
$SMRT_ROOT/admin/template/smrtlink.service.tmpl
```

Follow the instructions in the template comments to make site-specific modifications and install as a systemd service unit.

LDAP integration

SMRT Link supports integration with LDAP for user login authentication, as well as using local Keycloak users that exist **only** within SMRT Link.

If you are interested in configuring SMRT Link integration with your organization's LDAP, PacBio recommends that you consult your LDAP administrator to help determine the correct LDAP settings.

Note: Existing LDAP configurations are automatically migrated during upgrade.

Configuring LDAP in Keycloak

- LDAP is configured after SMRT Link v13.0 is installed, using the Keycloak authentication server software, as shown below.
- SMRT Link must **first** synchronize with your organization's LDAP objects before any directory accounts can be enabled and given a role to facilitate SMRT Link access.

- 1. Enter the following in your browser: https://<hostname>:9443/auth/admin/ where <hostname> is the host where SMRT Link is installed.
- 2. Login using admin/admin (unless you have changed the password).

	Sign in to your account
Username	or email
admin	
Password	

3. Under **Configure** on the left hand side of the window, click **User Federation**.

						🛓 Admin 🗸
SMRTLink ~	User Federa	tion				
Configure						Add provider 🗸
👫 Realm Settings		Enabled	Provider Name	Priority	Actions	
- Clients	Idap	true	Ldap	0	Edit	Delete
∼ & Client Scopes ≣ Roles	\smile					
Identity Providers User Federation						
Authentication						
Manage						
🏝 Groups						

- 4. On the right hand side of the window, click **Add provider...** and select the **Idap** option.
- 5. Enter the required fields (and any others necessary for your LDAP server) and verify that you can connect to the server using the **Test connection** and **Test authentication** buttons.

KEYCLOAK					
SMRTLink 🗸	User Federation > Add user storage provider				
Configure	Add user federation	provider			
👫 Realm Settings	Required Settings				
🗑 Clients 🗞 Client Scopes	Enabled 😡	ON			
Roles	Console Display Name 🚱	Idap			
	Priority 😡	0			
😂 User Federation	Import Users 😡	ON T			
Authentication	Edit Mode @				
Manage	Sync Registrations 🖗	OFF			
🚢 Users	* Vendor 🔞	v			
 Sessions 	* Username LDAP attribute 🖗	LDAP attribute name for username			
 Events Import 	* RDN LDAP attribute 😡	LDAP ottribute name for user RDN			
Export	* UUID LDAP attribute 😡	LDAP attribute name for UUID			
	* User Object Classes 😡	LDAP User Object Classes (div. by comma)			
	* Connection URL 😡	LDAP connection URL	Test connection		
	* Users DN 🚱	LDAP Users DN			
	Custom User LDAP Filter 🔞	LDAP Filter			
	Search Scope 🚱	One Level 🗸			

The following fields are **required**. (**Note**: Values provided in the example above are listed below for clarity. Actual values should be provided by your LDAP administrator):

- Username LDAP attribute: uid (if you are using Active Directory, the most likely value is sAMAccountName)
- RDN LDAP attribute: uid (usually the same as the Username LDAP Attribute)
- UUID LDAP attribute: entryUUID
- User Object Classes: person, organizationalPerson, user
- Connection URL: ldap://ldap.university:389
- Users DN: CN=users, DC=university, DC=edu
- (Optional) Custom User LDAP Filter: (objectClass=person)

If Bind Type is simple, you also need to enter credentials for accessing the directory:

- Bind DN: CN=ldapadmin, CN=users, DC=university, DC=edu (This is the user account that is used to authenticate to the LDAP environment.)
- Bind Credential: <password>
- 6. When you are finished click **Save**.
- 7. After you save the LDAP configuration, additional buttons display at the bottom of the window. Clicking **Synchronize all users** imports **all** users to the Keycloak database **without** assigning them SMRT Link roles.

Save Cancel Synchronize changed users Synchronize all users Remove imported Unlink users

8. Enable SMRT Link users individually as described in the next section.

For more information on LDAP, consult the following web pages:

https://en.wikipedia.org/wiki/Lightweight_Directory_Access_Protocol https://en.wikipedia.org/wiki/LDAP_Data_Interchange_Format https://msdn.microsoft.com/en-us/library/ms677605%28v=vs.85%29.aspx

Problems with the LDAP server may be debugged by looking at the log file located here:

\$SMRT_ROOT/userdata/log/smrtlink-analysisservices-gui/keycloak.stdout

Note: If LDAPS needs to be used, the appropriate SSL certificate needs to be installed in a format understood by Keycloak. Use keytool to add the LDAPS X.509-formatted public certificate to a JKS file named keycloak-truststore.jks, set the passphrase to password1, and enter yes to force trust when prompted. To install the keystore, simply copy it to this location:

\$SMRT_ROOT/userdata/config/security/keycloak-truststore.jks

Start the SMRT Link services, and in step 5 of the LDAP Integration instructions above, change the Connection URL to use an *ldaps:// URI* format, and, if necessary, adjust the port number. (Note: By default, LDAP uses TCP port 389 and LDAPS uses TCP port 636).

SMRT Link user roles

SMRT Link supports three user roles: **Admin**, **Lab Tech**, and **Bioinformatician**. Roles define which SMRT Link modules a user can access. The following table lists the privileges associated with the three user roles: PacBio recommends the following role assignments:

Tasks/privileges	Admin	Lab Tech	Bioinformatician
Add/delete SMRT Link users	Y	N	N
Assign roles to SMRT Link users	Y	N	N
Update SMRT Link software	Y	N	N
Add/update instruments	Y	N	N
Access Instruments module	Y	Y	Y
Access Sample Setup module	Y	Y	N
Access Runs module	Y	Y	N
Access Data Management module	Y	Y	Y
Access SMRT Analysis module	Y	Y	Y

- Assign at least one user per site to the Admin role. That individual is responsible for enabling and disabling SMRT Link users, as well as specifying their roles and adding/removing associated Revio instruments. The Admin can also access all SMRT Link modules, as well as every file in the system. (Note: SMRT Link supports multiple users with the Admin role per site.)
- Assign users who work in the lab preparing samples and performing runs the **Lab Tech** role. **Lab Tech** can also access all SMRT Link modules.
- Assign users who work **only** on data analysis the **Bioinformatician** role. **Bioinformatician** can **only** access the Instruments, Data Management and SMRT Analysis modules; this is the lowest access level.

Note: The Admin role only allows a user account to administer the configuration options available through the SMRT Link browser UI. It does **not** provide access to the Keycloak management interface, which is intentionally restricted to the built-in admin user **only**.

Adding SMRT Link users via LDAP integration and assigning user roles

- To enable users via LDAP integration, you must **first** configure LDAP **before** you can manage users and assign SMRT Link roles to users.
- After LDAP is configured, if you do not assign a SMRT Link role to a user, that user will not be able to login to SMRT Link.
- 1. Access SMRT Link: Enter https://<hostname>:8243/sl/home, where <hostname> is the host where SMRT Link is installed.
- 2. Choose **Settings > User Management** at the top of the page.
- 3. There are two ways to find users:
- To display all SMRT Link users: Click Display all Enabled Users.
- To find a specific user: Enter a user name, or partial name and click Search By Name.

Configuration	
Sections	User Management
Instruments	DISPLAY ALL ENABLED USERS
User Management	Q Search By Name

- 4. Click the desired user. If the Status is **Enabled**, the user has access to SMRT Link; **Disabled** means the user **cannot** access SMRT Link.
 - To add a SMRT Link user: Click the Enabled button, then assign a role. (See Step 5.)
 - To disable a SMRT Link user: Click the Disabled button.
- 5. Click the **Role** field and select one of the three roles. (A **blank** role means that this user **cannot** access SMRT Link.)
- 6. Click **Save Changes**. The user now has access to SMRT Link, based on the role just assigned.

	User Details	
-	User Name	
-	Administrator	
	Status	
	ENABLED	DISABLED
	Role	
	Admin	\$
	Contact Information	
	Email Address	
	Administrator@pacificbiosciences.co	m
-	Phone Number	
-	Phone	
		Cancel Save Changes

Adding local users to SMRT Link using Keycloak

SMRT Link is designed to integrate with an LDAP server to provide user account information, but it is also possible to add **local** user accounts using the Keycloak server that handles authentication for the API gateway.

To add a local account:

- 1. Access the Keycloak management GUI at https://<servername>:9443/auth/admin and log in with SMRT Link's built-in admin account credentials (admin/admin by default.)
- 2. On the left-hand menu, under Manage, click Users.

3. Click the **Add user** button on the right-hand side of the screen.

KEYCLOAK			
MRTLink	Users		
	Lookup		
詂 Realm Settings	Search	Q View all users	Unic
🕞 Clients	Please enter a sea	ch, or click on view all users	
🗞 Client Scopes			
Roles			
Identity Providers			
User Federation			
Authentication			
🐁 Groups			
💄 Users			

4. Complete the form and click **Save**.

SMRTLink 🗸	Users > Add user	
Configure	Add user	
👫 Realm Settings	ID	
📦 Clients	Created At	
🚓 Client Scopes	Username *	Pacbiouser
Roles	Fmail	someone@edu.com
Identity Providers	Lindi	somone@edd.com
User Federation	First Name	Pacbio
Authentication	Last Name	User
Manage	User Enabled 🚱	ON
k Groups	Email Verified 😡	OFF
💄 Users		
 Sessions 	Groups 😡	Select existing group
🛗 Events		No group selected
团 Import		
Export	Required User Actions 😡	Select an action
		Save

5. In the newly added user page, click **Credentials**, and enter a password for the user. If you are issuing a temporary password that the user needs to change on first login, make sure the **Temporary** toggle is **ON**. The section below covers password changes.

KEYCLO AK				
SMRTLink 🗸	Users > pacbiouser			
Configure	Pacbiouser 👕 🦯			
👫 Realm Settings	Details Attributes Cree	lentials Role Mappings Groups Co	onsents Sessions	
📦 Clients	Manage Credentials			
🚓 Client Scopes	Position	Type	liser Label	Data
identity Providers	1 Ostion	1300		Dutu
User Federation	Set Password			
Authentication	Password	••••••		۲
Manage	Password Confirmation			۲
ka Groups	Temporary 🕖			
Lusers		Set Password		
 Sessions 	Credential Reset			
· Events	eredentidi heset			
Export	Reset Actions 🖗	Select an action		
	Expires In 😡	12 Hours V		
	Reset Actions Email 🚱	Send email		

To change a temporary password:

1. When a new local user attempts to log in to SMRT Link with a temporary password, the login will fail with the message "Account is not fully set up", and a button displays to open the Keycloak user console.



2. Log in to Keycloak with the same credentials. The Keycloak UI will prompt the user to enter a new password.

	SMRTLINK
	Update password
A New Pass	You need to change your password to activate your account, word
Confirm.	lassword
	Submit

- 3. Once the password has been changed, return to the SMRT Link login screen and enter the new password.
- 4. Local users may change their passwords again by navigating directly to the Keycloak user account page at https://<hostname>:9443/auth/realms/SMRTLink/account/#/.



SMRT Link and SSL certificate procedures

SMRT Link v13.0 uses SSL (Secure Sockets Layer) to enable access via HTTPS (HTTP over SSL), so that your SMRT Link logins and data are encrypted during transport to and from SMRT Link. SMRT Link includes an authentication server (Keycloak), which can be configured to integrate with your LDAP/AD servers and enable user authentication using your organizations' user name and password. To ensure a secure connection between the SMRT Link server and your browser, a domain-specific SSL certificate may be installed **after** completing SMRT Link installation.

It is important to note that PacBio will **not** provide a CA-signed SSL certificate. However, once your site has obtained a CA-signed SSL certificate, PacBio's tools can be used to install it for use with SMRT Link web services. (**Note**: PacBio recommends that you consult your IT administrator about obtaining an SSL certificate.) You will need a certificate issued by a certificate authority (CA). PacBio has tested SMRT Link with certificates from the following certificate vendors: VeriSign, Thawte and DigiCert.

If your site does **not** provide an SSL certificate, SMRT Link v13.0 will use a PacBio self-signed SSL certificate. If you use the self-signed SSL certificate, **each** user will need to accept the browser warnings related to access in insecure environment. You can also have your IT administrator configure desktops to **always trust** the provided self-signed certificate. Note that SMRT Link is installed within your organization's secure network, behind your organization's firewall.

See "Using SMRT Link with a PacBio self-signed SSL certificate" on page 22 for details on how to handle the security warnings when accessing SMRT Link.

Use the following procedures **only** if your site provides an SSL certificate. These procedures are **not** applicable if you are using PacBio's self-signed SSL certificate.

Note: If you have **already** setup an SSL certificate in SMRT Link v4.0.0 or later, those settings will **not** be carried over **automatically** when migrating from WSO2 API Manager to the new API gateway.

Installing an SSL certificate for NGINX

In the new API gateway, SSL transport is handled by the NGINX web server, which uses a simpler configuration consisting of a plain-text certificate and private key. By default SMRT Link will generate a self-signed certificate and key the first time you start the new API gateway:

```
$SMRT_ROOT/userdata/config/security/pb-smrtlink-default.crt
$SMRT ROOT/userdata/config/security/pb-smrtlink-default.key
```

To install a custom certificate for NGINX

 Stop SMRT Link services: SMRT_ROOT/admin/bin/services-stop

- 2. Copy the certificate and private key files to these paths: \$SMRT_ROOT/userdata/config/security/smrtlink-site.crt
 \$SMRT_ROOT/userdata/config/security/smrtlink-site.key
- 3. Start SMRT Link services:
 \${SMRT_ROOT}/admin/bin/services-start

Restoring the default self-signed SSL certificate

It may sometimes be necessary to uninstall the user-provided SSL certificate and restore the default certificate. All that is needed for this is to do the opposite of the install steps and remove or rename the site certificate.

Using SMRT Link with a PacBio self-signed SSL certificate

SMRT Link v13.0 uses self-signed SSL certificate generated by the installer. If your site does **not** have a signed SSL certificate **and** you use the self-signed SSL certificate, **each** user will need to accept the browser warnings related to access in insecure environment. You can also have your IT administrator configure desktops to **always trust** the provided self-signed certificate. Note that SMRT Link should be installed within your organization's secure network, **behind** your organization's firewall.

Security messages display when users try to login to SMRT Link for the **first time** using the Chrome browser. These messages may also display **other times** when accessing SMRT Link. **Each** SMRT Link user in your organization should address these browser warnings following the procedure below.

1. The first time you start SMRT Link after installation, you see the following text. Click the Advanced link.



2. Click the Proceed... link. (You may need to scroll down.)



3. Close the window by clicking the **Close** box in the corner.



4. The **Login** dialog displays, where you enter the User Name and Password. The next time you access SMRT Link, the Login dialog displays **directly**.

Migrating from WSO2 API Manager

If you are upgrading SMRT Link from an installation that used WSO2 API Manager for secure access and user accounts, you need to migrate your configuration to the replacement API gateway **before** you can use new features, such as support for Revio systems. Although some parts of this process are automated, re-entry of some account information (such as passwords) is usually required, and SSL certificates need to be manually imported as described in the previous section.

The migration can be initiated when you first start SMRT Link:

\$SMRT_ROOT/admin/bin/services-start -migrate

When the server has fully started, an interactive script is launched to take you through the migration steps, including the configuration of LDAP server(s) if used, and migration of user account settings. The migration script will attempt to extract the essential fields from existing WSO2 LDAP configurations and register them with Keycloak; if this step is run you will be prompted to re-enter the connection password(s).

If any step in the automated migration fails, you can always complete the configuration manually using the Keycloak administrative interface as described previously.

Because the cleartext passwords for local user accounts are not available, these accounts will **not** be initially available after migration **until** you reset their passwords. See the previous section on creation of local users and setting temporary passwords for instructions on how to re-enable these accounts.

Importing data into SMRT Link

If you have a Revio system/Sequel II system/Sequel IIe system installed and it is linked to the SMRT Link software during the instrument installation, your data will be **automatically** imported into SMRT Link.

You can **manually** import the following types of files directly, using the Data Management module of the SMRT Link GUI:

- Subreads: XML file (.subreadset.xml) or ZIP file containing information about subreads from Sequel II and Sequel IIe systems, such as paths to the BAM files. Use only ZIP files created by SMRT Link.
- **HiFi reads**: XML file (.consensusreadset.xml) or ZIP file containing information about HiFi reads (reads generated with CCS analysis whose quality value is equal to or greater than 20.) Use **only** ZIP files created by SMRT Link.
- **Barcodes**: FASTA (.fa or .fasta), XML (.barcodeset.xml), or ZIP files containing barcodes.
- References: FASTA (.fa or .fasta), XML (.referenceSet.xml), or ZIP files containing a reference sequence for use in starting analyses.
- Target regions: BED (.bed) files that specify target genes or regions for analysis; used with the HiFi Target Enrichment and HiFi Mapping applications.

You can also import data in SMRT Link using the pbservice command-line utility, as shown below.

- The host and port for the analysis services are optional and default to localhost:8070. You can change
 these settings using the --host and --port arguments or by using the \$PB_SERVICE_HOST and
 \$PB_SERVICE_PORT environment variables.
- If using the authenticated port 8243, valid user credentials must also be supplied. Use the --user and --password switches (or --ask-pass), or set the \$PB_SERVICE_AUTH_USER and \$PB_SERVICE_AUTH_PASSWORD environment variables to specify the credential details.

Importing	Commands
BAM Data Sets	Import individual SubreadSet XML files:
generated by the Sequel II/IIe systems	<pre>\$> pbservice import-datasethost \$HOSTport \$PORTuser \$USERask-pass /path/to/subreads.subreadset.xml</pre>
	Import a directory of SubreadSet XML files:
	<pre>\$> pbservice import-datasethost \$HOSTport \$PORTuser \$USERask-pass /path/to/tree/containing/subreadssets.xml/</pre>

Sending log files to Technical Support

Troubleshooting information can be sent to PacBio Technical Support multiple ways. If there is a connection to the PacBio Event Server, do the following:

- From the SMRT Link menu: About > Troubleshooting Information > Send.
- From a SMRT Link "Failed" analysis Results page: Click Send Log Files.

If there is connectivity to the PacBio Event Server, run the following command to generate the information and automatically send it to PacBio Technical Support:

\$SMRT_ROOT/admin/bin/tsreport-install --bundle --upload

If there is **no** connectivity to the PacBio Event Server, run the following command to generate a .tgz file and email the file to support@pacb.com to file a case:

```
$SMRT_ROOT/admin/bin/tsreport-install --bundle
```

The generated file can be found here: \$SMRT_ROOT/userdata/tsreport/data/ts-install.tgz.

Note: The SMRT Link logs archive bundle will be limited to logs from approximately the past 24 hours. Ensure the above tsreport-install options and SMRT Link menu's **Send** button are run within **one day** of experiencing the issue being addressed.

Appendix A: SMRT Link workflow terminology

Management of all SMRT Link activity is handled by the SMRT Link services. In SMRT Link v8.0 and later, the pbsmrtpipe workflow engine was replaced by Cromwell, an open-source engine developed by the Broad Institute (https://cromwell.readthedocs.io/en/stable/). A continually-running Cromwell server is launched at the same time as SMRT Link services, which executes all jobs directly without spawning new processes. Several user-configurable settings control the use of compute resources by Cromwell. A representation of the SMRT Link services hierarchy is shown below.



NWORKERS: A SMRT Link services setting that specifies the maximum number of simultaneous analysis jobs (or workflows, as Cromwell refers to them) that may be run.

CONCURRENT_JOB_LIMIT: A Cromwell configuration setting that limits the total number of job submissions to a specific backend, across all running workflows.

MAXCHUNKS: A Cromwell workflow that limits the maximum number of pieces a large Data Set may be broken into for parallelized analysis.

NPROC (Not shown in diagram): A Cromwell workflow setting that limits the maximum number of slots that any single JMS cluster submission may request.

Appendix B: Distributed computing setup

PacBio supports the following Job Management Systems (JMS): **Sun Grid Engine (SGE)**, **PBS**, **LSF**, and **SLURM**. You may attempt to manually configure for alternate job management systems, but these are **not** guaranteed to work.

A Job Management System may be used to dispatch jobs to a distributed compute environment. If **no** Job Management System is specified, the system will run in non-distributed mode, and **all** compute jobs will be run locally on the install host.

Available Job Management Systems are detected from the PATH environment variable, but may also be selected manually.

For more information on customizing all of the submissions to the Job Management Systems, see the comments in the file <code>\$SMRT_ROOT/userdata/user_jmsenv/user.jmsenv.ish</code>. Note that changes to this file will apply to **every** job submitted to the cluster.